

Laser Pinsetter



High Performance, Heavy Duty Laser Pinsetter From A Proven Leader in Laboratory Products and Supplies

The Buffalo Dental Laser Pinsetter provides the ideal method for drilling dowel pin holes in dental models. Precise drilling location is displayed by a state-of-the-art laser beam positioned above the worktable and in perfect alignment with the drill bit.

To use the system, place the model flat on the worktable and determine the desired location for the dowel pin. Then by pushing down on the model, the high speed drilling system is automatically activated and the correct hole is drilled up into the underside of the model. No more questions about dowel pin location! No more cumbersome activation switches!

The Pinsetter is easy to use in an upright or sit-down position, has adjustable drilling depth, and accommodates a variety of standard pin drills.

The unit comes ready to use and includes a standard, counter-sunk, "P" type drill.

- **Upright or Sit-Down Use-** Ergonomic design allows operation in convenient standing or seated position
- **Automatic High Speed Drilling-** Easy to use and allows for productive, high volume pin drilling w/ 1/8" ID Chuck
- **Laser Light Beam Alignment-** Indicates exact location of drill bit and assures precision
- **Heavy Duty Construction-** Protects drill motor and helps provide a long working life

● **Ordering Information**

<i>Item #</i>	<i>Description</i>
39000	Laser Pinsetter Complete, 120V AC, Including 1/8" ID Chuck and No. 1 Std. P-Type Drill
39005	Laser Pinsetter Complete, 220V AC, Including 1/8" ID Chuck and No. 1 Std. P-Type Drill
39100	Pinsetter Drills No. 1, Pkg. of 2 (w/ 1/8" Shank Diameter)

Notes: For laboratory use only. Use extreme caution and wear personal protective eyewear, gloves and equipment during operation. Avoid personal contact with the spinning bur, and always unplug machine when changing burs or cleaning the housing.



Made in USA!



Can also be used in an angled position for convenient seated operation!

Buffalo Dental Manufacturing Co., Inc.,
159 Lafayette Dr., PO Box 678, Syosset, NY 11791-0678 USA
Tel: 516.496.7200 or 800.828.0203 **Fax:** 516.496.7751
www.buffalodental.com

S39000 0708

***Please order today from
your favorite Dealer!***